Quick Start Guide

BigKeys LX

Updated 02/25/20

Getting Started

Connect BigKeys LX to a USB port on the device you will be using.

Cursor Keys

To use these special cursor control keys, hold down the F key and the arrow pad keys become PgUp, End, PgDn and Home.

Function Keys

This feature lets you access the Function keys F1 - F10. Simply hold down the F key and the numeric row becomes F1 through F10.

Special Characters

[F with ;
]	F with ‘
{	F with :
}	F with “
F11	F with /
F12	F with `
€	F with E
Prt Scrn	F with P

Assist Mode

The ASSIST MODE accommodates those who cannot press two (or more) keys simultaneously. ASSIST MODE works with and enhances the Windows “Sticky Key” Accessibility Option. When not in ASSIST MODE, the function keys are produced by pressing simultaneously the F key and a number key. (F+1 produces the F1 function, etc.)

Assist Mode enables you to press the F key and have it remain active until you press another key. In ASSIST MODE, the F key becomes a “Sticky Key”.

Examples of using ASSIST MODE:

- To obtain an F1 in ASSIST MODE, press and release the F key, then press and release the 1 key.
- To obtain an “Alt-F4” in ASSIST MODE, press and release the Alt key, then press and release the F key, then press and release the 4 key. The Windows “Sticky Keys” Accessibility Option must be turned ON for this example.

Assist Mode Continued



To enter ASSIST MODE press the CAPS LOCK key three times in a row. To exit ASSIST MODE and return the keyboard to normal operation, again press the CAPS LOCK three times in a row. Note that while in ASSIST MODE it is necessary to press the F key twice to send an F to the computer.

No Run-on Option

BigKeys LX is designed to allow only one keystroke to be sent to the computer for each key depression. This feature prevents multiple characters that might occur inadvertently when holding down a key too long. See Set Options below to turn the No Run-On Option off.

QWERTY and ABC Layouts

BigKeys LX offers a QWERTY and ABC layout. By removing and rearranging the key caps, you can reprogram your BigKeys LX to use either layout. See Set Options below to change from QWERTY to ABC or ABC to QWERTY.

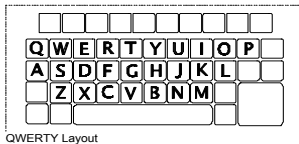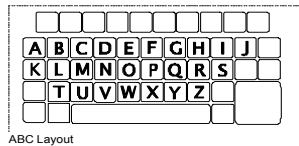Set Options

Be sure the computer is powered on and that the BigKeys is unplugged. Hold down the numeric key for the option desired. While holding down the key, plug the keyboard into a USB port on the computer. Release the key after 5 seconds, the option will now be in place.

- QWERTY Mode use numeric key 1
- ABC Mode use numeric key 2
- No-Run-On use numeric key 3
- Run-On use numeric key 4

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113
(800) 322-0956 / www.ablenetinc.com

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For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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FCC

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.

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Warning: Reorient or relocate the receiving antenna.
Increase the separation between equipment and receiver.
Consult AbleNet Technical Support or your local Distributor for help.

Warning: Reorient or relocate the receiving antenna.
Increase the separation between equipment and receiver.
Consult AbleNet Technical Support or your local Distributor for help.

Warning: Reorient or relocate the receiving antenna.
Increase the separation between equipment and receiver.
Consult AbleNet Technical Support or your local Distributor for help.

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