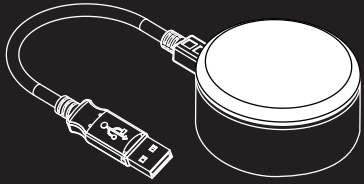
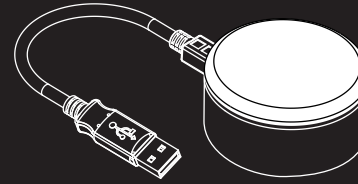
## Quickstart Guide: Switch Click™ USB

**AbleNet®**  
www.ablenetinc.com

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**AbleNet®**  
www.ablenetinc.com

### Description

The Switch Click USB emulates a mouse or keyboard click (default left mouse click) to a computer when activated. The device has a USB connection and switch input jack allowing you to attach your favorite switch to the Switch Click USB.

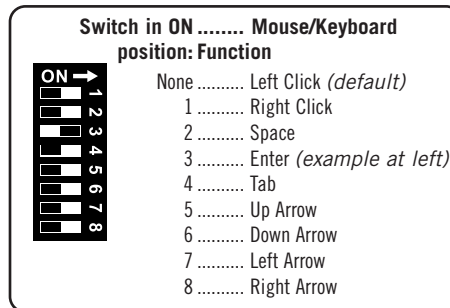
### Basic Operation:

1. Connect the Switch Click USB to any available USB port on your computer.
2. Press the switch top to send a left-click to your computer.

### Configuring Device Output:

The Switch Click USB can send one of several signals to a computer. To select the desired signal:

1. Ensure that the Switch Click USB is not plugged into a computer.
2. Remove the bottom panel by loosening the screws.
3. Configure the dip switches as shown at right.
4. Replace the bottom panel and plug the Switch Click USB into a computer.

For best results use the Switch Click USB with scanning software. *Switch Click USB will not move a cursor or arrow around the screen.* Configure any scanning software to recognize the same signal as the Switch Click USB.

### Using an External Switch:

The switch jack on the side of the device allows you to connect any single switch (mono 1/8" plug). This jack performs the same function as the switch top. It does not add a second click behavior.

### Troubleshooting

If you have problems with the Switch Click, first ensure that any scanning software is correctly configured. Also consider opening your computer's control panel software, examining mouse functions.

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## Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113  
(800) 322-0956 / [www.ablenetinc.com](http://www.ablenetinc.com)

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## For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

## For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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