# AbleNet®

## BIG TalkingBrix™

www.ablenetinc.com  
(800) 322-0956

### Message Recording

1. On back of BIG TalkingBrix, move the switch to REC
2. Press and hold colored switch top
3. Begin speaking for up to 10-seconds when light starts blinking
4. Release colored switch top when finished
5. On back of BIG TalkingBrix, move switch from REC to ON to start using

A picture symbol up to 3-in / 77-mm can be used with BIG TalkingBrix. Lift clear snap switch cap off of colored switch top to insert picture symbol. Re-attach clear snap cap to hold picture symbol on BIG TalkingBrix.

### Multi-Message Communication

You can connect multiple BIG TalkingBrix together to create a multi-message communication device. Connect using the tabs and slots found on the side of each BIG TalkingBrix.

### Mounting

BIG TalkingBrix include internal magnets for mounting. Or, use hook-and-loop fastener on the back of BIG TalkingBrix.

### Charging

When BIG TalkingBrix has a low battery, the light will blink red three times after message playback. Attach BIG TalkingBrix to the included wall charger. The light will be solid red when charging and solid green when fully charged. BIG TalkingBrix cannot be used while charging.

### Safety Shut-Off

The BIG TalkingBrix battery contains a safety feature that temporarily disables the device if the battery reaches 113-degrees Fahrenheit / 45-degrees Celsius. BIG TalkingBrix will continue to work once the battery temperature decreases below 113-degrees Fahrenheit / 45-degrees Celsius.

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## Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at [www.ablenetinc.com](http://www.ablenetinc.com).

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113  
(800) 322-0956 / [www.ablenetinc.com](http://www.ablenetinc.com)

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## For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product, to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

## For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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