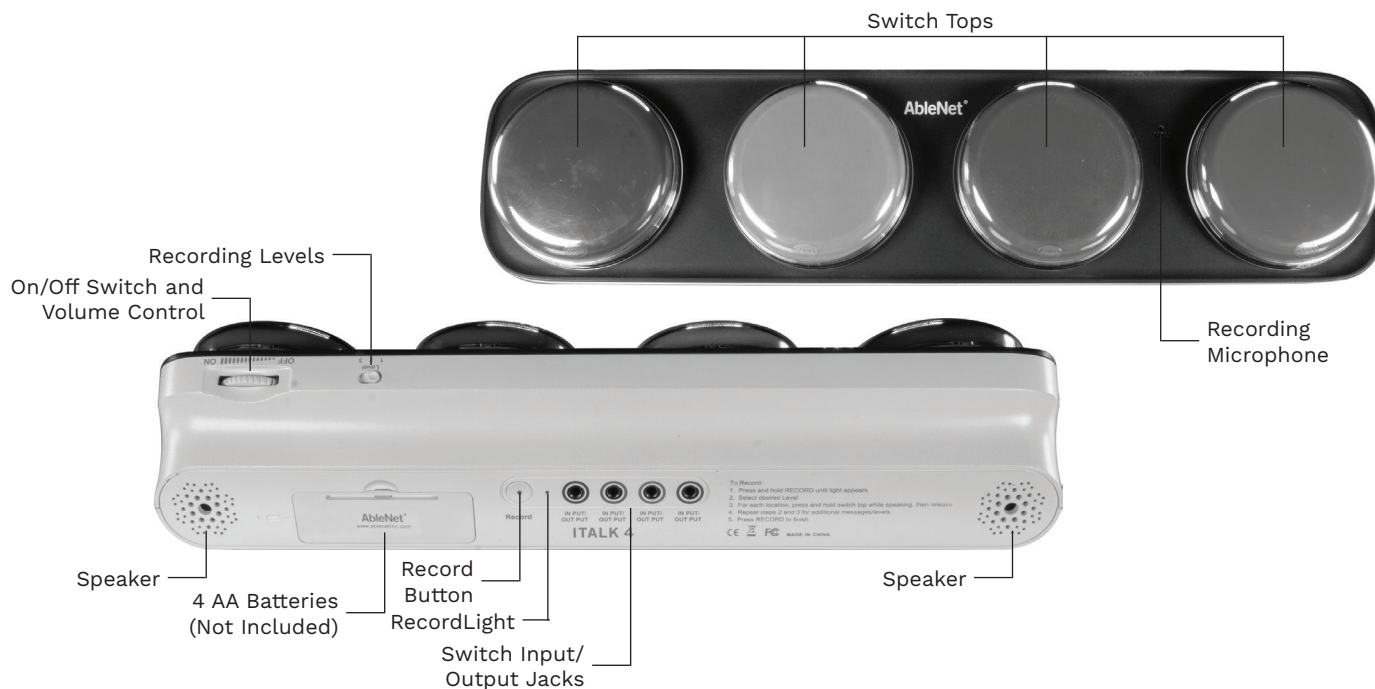
AbleNet[®]

iTalk4[™]

www.ablenetinc.com
(800) 322-0956

Getting Started

1. Insert 4 AA batteries (not included)
2. Turn iTalk4 On
3. Press and hold the record button until the record light activates
4. Press and hold a switch top. Begin speaking into recording microphone when you hear a beep. Release when finished recording message.
5. Repeat for remaining switch tops
6. Press and release record button to exit record mode.
7. Optional: Select a different recording level to record a second and third set of messages.
8. iTalk4 is now ready to use. Press and release desired location to playback message

Recording Levels

iTalk4 provides three recording levels for increased flexibility.

Use the toggle switch on the top of iTalk4 to select the appropriate recording level. You can record separate messages for each message location on each level.

Controlling Toys and Appliances

The iTalk4 switch input/output jacks allow you to activate switch adapted toys or appliance with this device.

1. Plug any switch-adapted toy or appliance into any of the switch input/output jacks
2. Record a message into the corresponding location
3. Press the switch top to play the message and activate the attached toy or appliance. The toy or appliance will turn on for the length of the message.

Mounting

Use the AbleNet Universal Mounting Plate (sold separately) hole pattern D to mount iTalk4 to any of AbleNet's mounting systems. Threaded mounting holes are located on the bottom of iTalk4.

Playing A Message With An External Switch

iTalk4's switch input/output jacks default to controlling toys and appliances. To use external switch(es) with iTalk4:

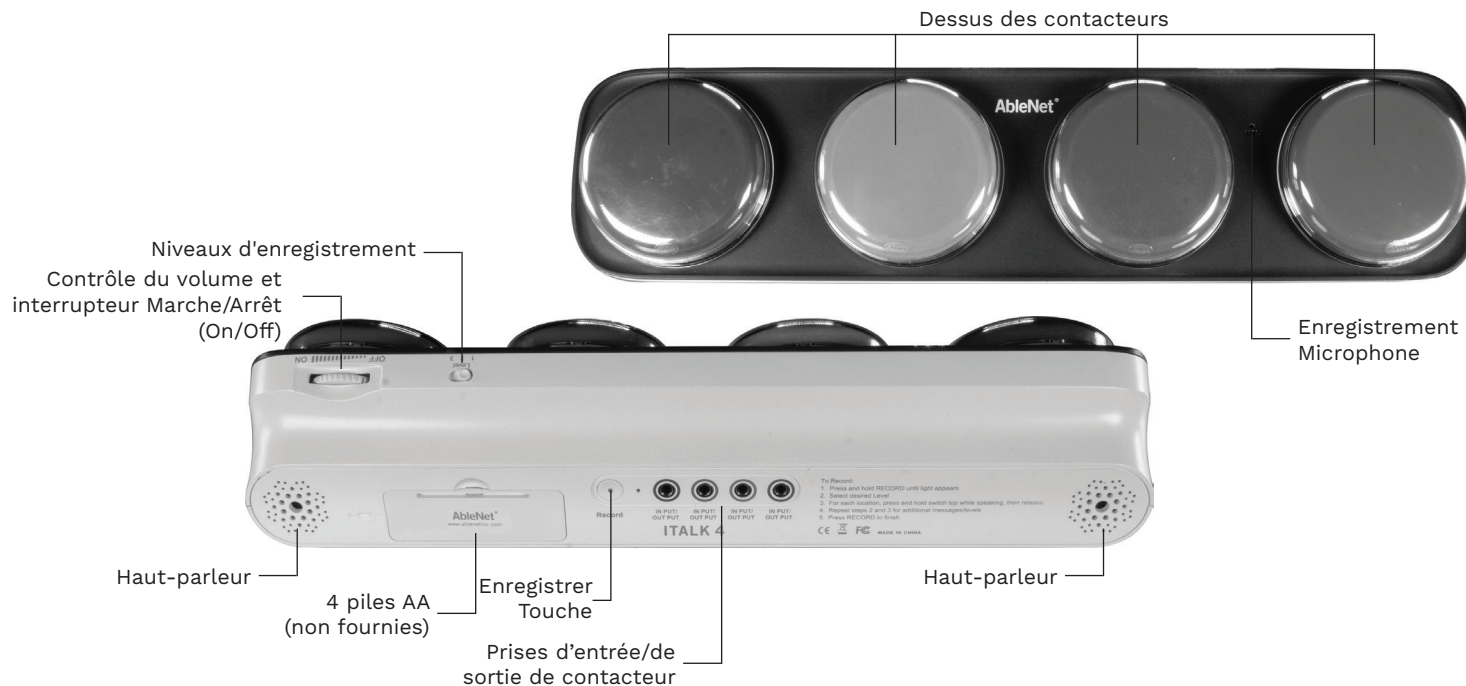
1. Turn iTalk4 off
2. Attach switch(es) of your choice with 1/8-inch (3.5mm) plugs into the appropriate switch input/output jack(s)
3. While holding any external switch down, turn iTalk4 on

The toy/appliance control feature is disabled when using an external switch(es). To revert to default toy/appliance function, turn iTalk4 off and back on.

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113
(800) 322-0956 / www.ablenetinc.com

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For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product, to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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