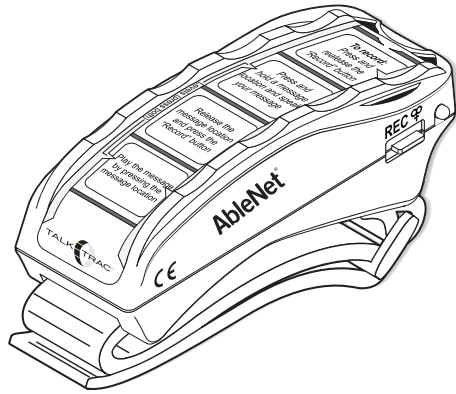
Quickstart Guide

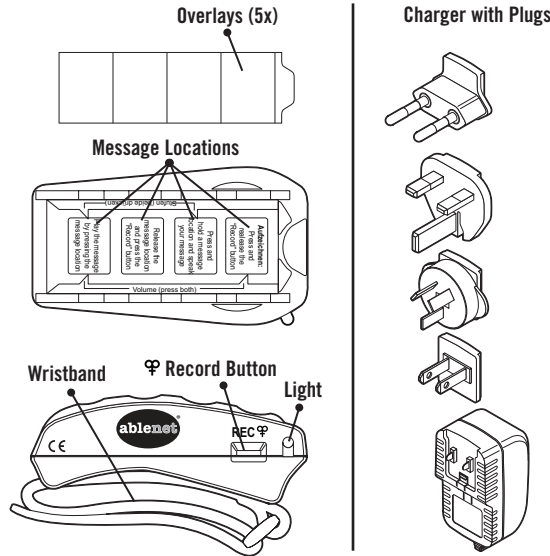
TalkTrac™ Wearable Communicator

AbleNet®


- Eight - 10 second messages
- Two levels of 4 messages
- Two volume levels
- Rechargeable

Overview

NOTE: The wristband can be replaced with standard wristbands.

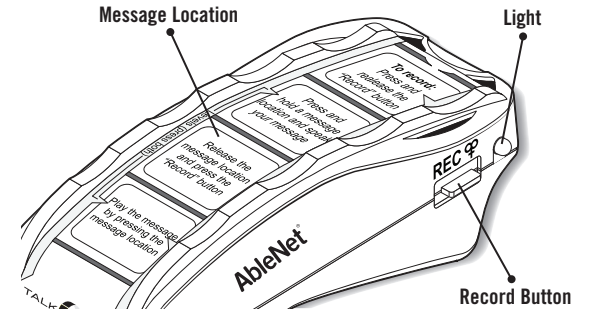Record and Play Messages


To Record a Message

1. Press and hold the Record button until light turns red.
2. Press and hold a message location (light flashes red) and speak message.
3. Press and release the Record button (light off).

To Play a Message

Press and release the message location.

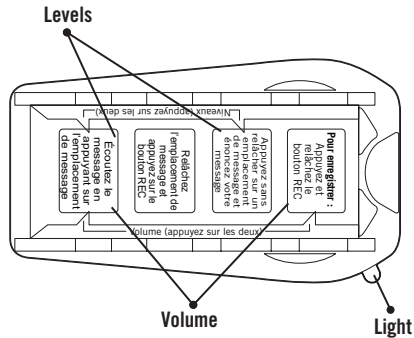Volume and Changing Levels


To Change Volume

Press both volume locations shown.

To Change Levels

Press both level locations shown (green light will flash the level, 1 or 2).

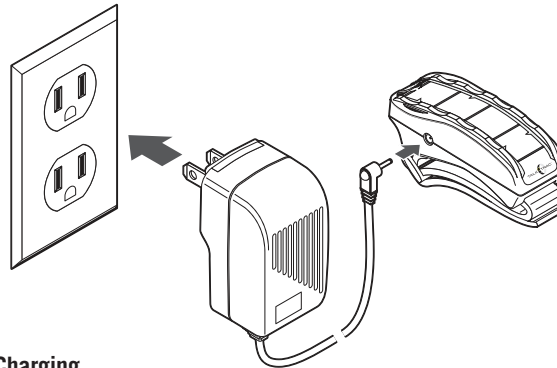Charging and Low Battery

Charging

Charge each night to maintain a full charge.

NOTE: Charge for 24hrs before first use. TalkTrac will not work while charging.

Low Battery Warning

TalkTrac will flash red multiple times after a message play. The battery level is getting low and should be charged soon.

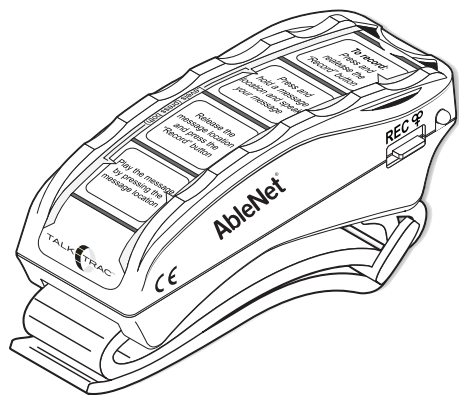
Overlays

Overlays

Additional overlays and can be created using Mayer-Johnson® Boarkmaker or by downloading a blank template from AbleNet's website.

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113
(800) 322-0956 / www.ablenetinc.com

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For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product. to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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