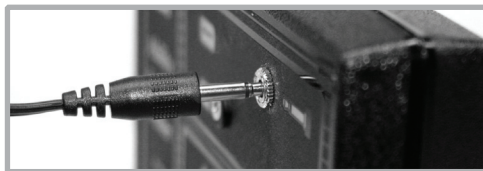
Quickstart Guide: Single, Dual and Multiple switches.

ablenet | **tash**
an ablenet company
www.ablenetinc.com

Connecting a switch to a device

Single Switches

(Big Red®, Jelly Bean®, Buddy Button, et al.)

Plug a single switch into any switch-adapted device with a standard mono 3.5mm switch jack.

Dual Switches

(Rocker, Pneumatic switch)

Plug a dual switch into any switch-adapted device with a standard stereo 3.5mm switch jack.

Multiple Switches

(Star, Wafer, Penta, Mini Joystick, et al.)

Plug a multiple switch into any device with a compatible 9-pin port.

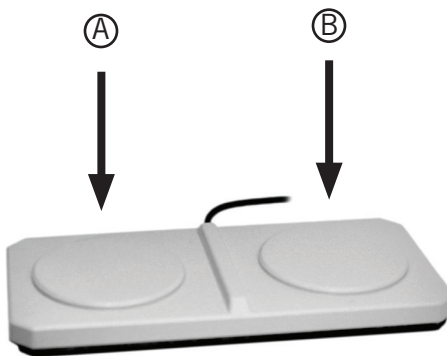
Note: For computer access a computer switch interface is required. (sold separately)

Activating a Single switch

After the single switch has been connected, press the activation area to control the attached device.

Activating a Dual switch


After the dual switch has been connected, press either activation area to control the attached device. Each side will act as an individual switch.

Example shown: Press side "A" to navigate through choices and side "B" to activate.

Note: Specific behaviors will vary by attached device. See manufacturer's documentation of the attached device for more information.

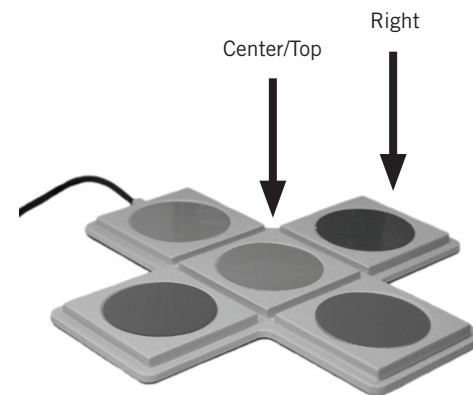Activating a Multiple switch

After the multiple switch has been connected, press any activation area to control the attached device. Each activation area will act as an individual switch.

Example shown: Press right side to navigate through choices and the press the center or top (if using a joystick) to activate.

Note: Specific behaviors will vary by attached device. See manufacturer's documentation of the attached device for more information.

Changing a Twist Cap*

* Available on Big Red and Jelly Bean Twist only.

Firmly, rotate the colored switch top counter-clockwise to remove. Attach a new color by rotating the switch top clockwise until it stops.

A Snap Switch Cap is sold separately for the Big Red and Jelly Bean switches. Place a picture or symbol on the switch top, and secure by snapping your Switch Cap over it.

Mounting

The Big Red, Jelly Bean and Specs switches:

The Big Red, Jelly Bean or Specs switch may be mounted onto the Slim Armstrong® and Universal Switch Mounting Systems using the appropriate mounting plate.

- Big Red – Large Triangular Mounting Plate
- Jelly Bean – Small Triangular Mounting Plate
- Specs – Small Circular Mounting Plate

The Specs switch also includes a space-saving *flush* base as well as a *strap* base allowing it to be worn or secured with a Velcro® strap.

For String Switch and other flat-surface mounting options, use UltraStik® reusable adhesive (for horizontal) and Dual Lock™ interlocking tape for vertical or angled surfaces.

Other Single and Dual Switches:

Single and Dual switches other than those listed above may be mounted on the Tash Mounting Arm, Gooseneck, or Universal Table Mount. Visit www.ablenetinc.com to find the appropriate mounting connection for your switch.

Multiple Switches:

Mount the Mini Joystick with Push using the U-Clamp available at www.ablenetinc.com. For other multiple switches, consider UltraStik or Dual Lock to secure on flat surfaces such as tabletops and trays.

Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text is available at www.ablenetinc.com.

Guarantee

AbleNet offers a 30-day money back guarantee if our products do not meet your needs or expectations. Products must be in new condition to honor this service. **Please follow our repair/return policy when sending products back under our guarantee.**

AbleNet, Inc. Repair Information

If documentation identifying the date of purchase is not supplied, the warranty period will be determined as two years from the manufacture date stamped inside the product.

All products under warranty are repaired free of charge. Products no longer covered by warranty may be repaired for a nominal fee. All repairs are warranted for 90 days. Before sending in a product for repair, please contact an AbleNet customer representative for the Return Authorization Number (RA#) and repair charge, if applicable.

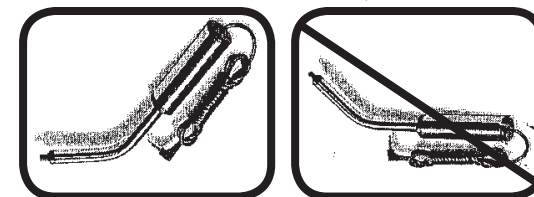
Call (800) 322-0956 for a Return Authorization Number

For more information on AbleNet products and services, contact:

AbleNet, Inc.,
Minneapolis/St. Paul, MN
800-322-0956 (US & Canada)
Internet: www.ablenetinc.com

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Información sobre operaciones de AbleNet, Inc.