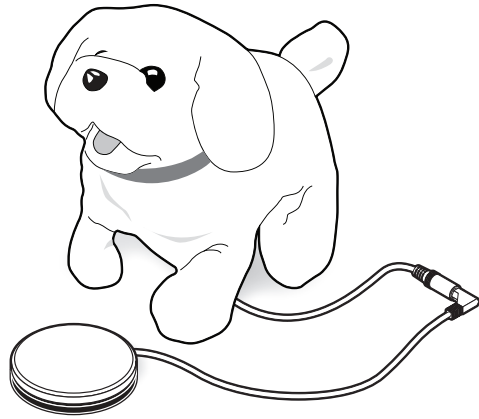
Quickstart Guide Switch Adapted Toys

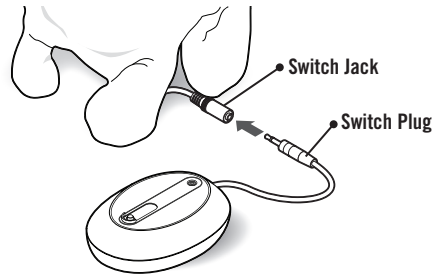
Thank you for purchasing a Switch Adapted Toy from AbleNet.


www.ablenetinc.com

Using the Toy

1. Insert a 3.5mm switch plug into Switch Jack.

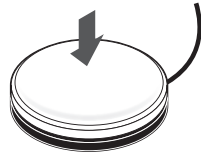
2. Turn product power ON
3. Press and hold switch to activate toy.


Changing the Batteries

For products with a Battery Device Adapter (BDA)

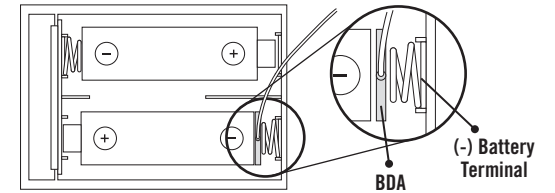1. Place the BDA between the neg (-) battery terminal and the neg (-) battery contact.

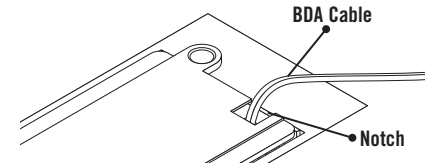2. Align cable in Notch and close battery door.

Controlling the Toy

Direct Control - Corded Switches

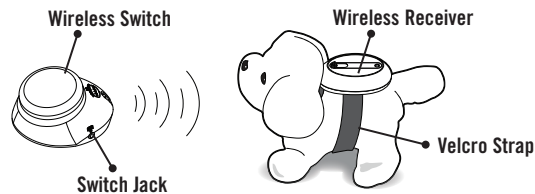
Press to turn ON and release to turn OFF.

See AbleNet website for a large selection of switches.

Wireless Control

Using a Wireless Switch and Receiver you can remotely activate the toy.


Attached the Wireless Receiver to the toy with a Velcro Strap.

Note: Any switch become wireless by using the Switch Jack on the wireless switch.

Latched or Timed Control

Using a Switch Latch and Timer with a switch will allow the toy to stay activated without holding the switch.

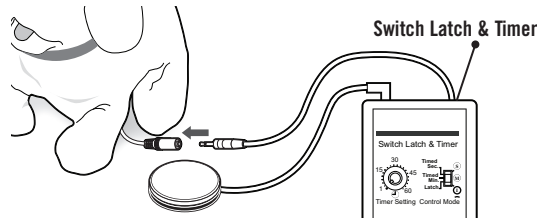Latched - Press switch to turn the toy ON. Press the switch again to turn the toy OFF.

Timed Second and Minutes - Adjust the dial to select the amount of time the toy should stay on. Press switch to activate for the selected duration.

Note: Wireless Latch and Timed control is also available using the Wireless SLAT.

Products and Accessories

Switch Latch & Timer	100SSLAT	JellyBeamer	10034200
Wireless Receiver	10034400	BigBeamer	10033700
Wireless SLAT	10034500	Velco Strap	80040100

Warranty and Contact Information

Warranty

Toy warranty listed on packaging or 90 days.

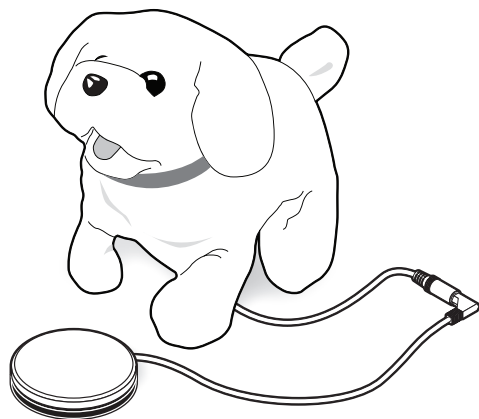
For more information on AbleNet products and services, or for contact:

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AbleNet, Inc.