

Subscription App Setup Instructions

Connect to Wi-Fi

1. On the QuickTalker Freestyle device, go to the *Utilities folder* and open the *Settings app*.
2. Tap on *Wi-Fi*
3. Tap on your Wi-Fi from the list of available Wi-Fi access points
4. Enter the password or complete any sign-on pop-ups that may appear.

Create the QuickTalker Freestyle Apple ID

1. In the *Settings app*, tap *Sign in to your iPad*. You will be prompted to enter an Apple ID. Tap *Don't have an Apple ID or Forgot it?*
2. Enter the user's First and Last name. **If the user is under 13, use 1/01/11 as the Date of Birth.** Users under 13 are not allowed to download apps from the App Store.
3. Next, create a new email address by selecting *Don't have an email address?*, then *Get an iCloud Email Address*. You will now be prompted to create an Apple ID. Enter your preferred Apple ID.
 - a. Note: The QuickTalker Freestyle Apple ID requires a new email address, which is only used for this device.
4. Create a password.
5. Enter a phone number you can use to verify the account. Then, use the code you receive to verify your account.

Download the Subscription App

1. On the QuickTalker Freestyle device, go to the *Utilities folder* and open the *App Store app*.
2. After dismissing any pop-ups, tap *Search* at the top
3. Search for *TD Snap*
4. Tap *Get*
5. Enter a billing address if it prompts for one, ensuring you select none for the payment method.

Instructions continued on next page →

TD Snap® is a trademark of Tobii-Dynavox LLC.

Subscription App Setup Instructions Continued

Redeem Gift Card Code(s)

1. Open the email you received to the email address you specified in the Self-Service form. It will be titled *ableCARE via Tremendous*. If you do not see the email, check your junk or spam folder.
2. Click redeem in the email
3. On the following webpage, click redeem
4. Select Apple
5. Click Redeem
6. On the next webpage, click view code
7. The final webpage should have the gift card code
8. On the QuickTalker Freestyle device, go to the *Utilities folder* and open the *App Store app*.
9. Open the Apple account.
 - a. The Apple account icon in the screen's upper right corner will differ depending on the Apple ID. The icon may look like initials, a blue silhouette of a person, or a profile picture. Tap that button.
10. Tap *Redeem gift card or code*.
11. Tap *enter code manually*.
12. You should now be greeted with a successful redemption page. Be careful to tap the *X* or *no thanks* on the next page as it will be an advertisement.

Activate Subscription

1. On the QuickTalker Freestyle device, go to the home screen of the device and open the *TD Snap app*. If the app is still downloading, wait until it is done downloading before opening it.
2. Complete the initial setup of the app
 - a. Answer the prompt according to what is needed for the client.
 - b. You will not need to log in to a TobiiDynamox account. There should be an option to skip in the lower right at this step.
3. Once you have completed the app setup, tap on the *Speech Bubble with a line going through it* in the top right of the screen.
4. Tap *1-month free – Subscribe* in the lower right of the pop-up.
5. You will be prompted to enter the *Apple ID Password*
6. You may be prompted to enter a billing address. Enter the address, ensuring that you select *None* for the payment method.
7. You should now be set up! Please notify ableCARE at ablecare@ablenetinc.com or (651) 294-3101 that you have completed the setup successfully. They will proceed to lock the device so the allocated funds for the subscription cannot be miss spent.

TD Snap® is a trademark of Tobii-Dynamox LLC.