## QUICK START GUIDE

# LITTLE Candy Corn™

accessibility switch

### WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at [www.ablenetinc.com](http://www.ablenetinc.com).

AbleNet, Inc.  
2625 Patton Road  
Roseville, MN 55113  
United States of America  
(651) 294-3101  
[ablecare@ablenetinc.com](mailto:ablecare@ablenetinc.com)  
[www.ablenetinc.com](http://www.ablenetinc.com)

Made with recycled content

EN June 18, 2021 1:57 PM

## 1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

## 2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

### To get started:

1. Connect switch plug to a device with a 3.5-mm / 1/8-in TS (mono) jack.
2. Turn switch on.
3. Hold hand or other part of body within 10-mm / 0.4-in of the activation area to activate your device.

*This device can do more! Full Instructions for Use are available at [www.ablenetinc.com](http://www.ablenetinc.com).*

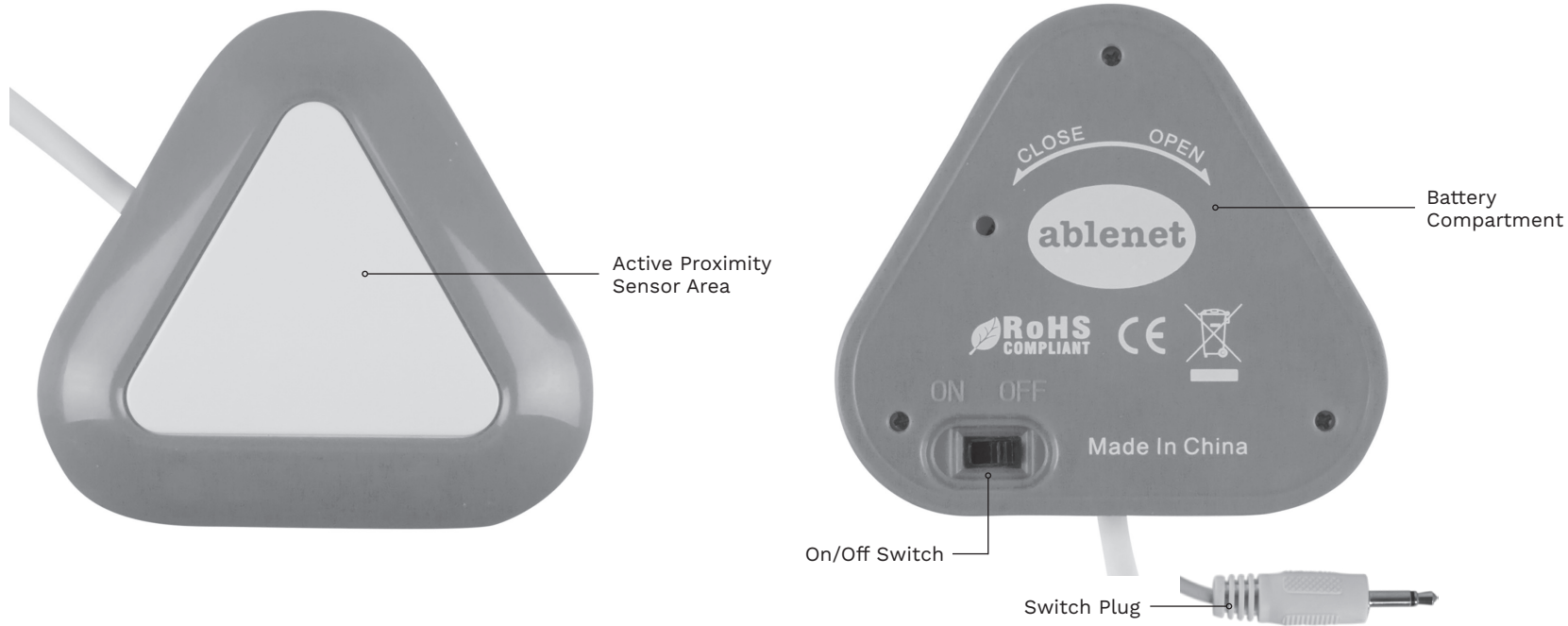
ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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