QUICK START GUIDE

SuperTalker™ FT

speech device

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Add four alkaline AA batteries to device (batteries not included).
2. Select and attach message grid for 1, 2, 4, or 8 message locations.
3. Turn device on.
4. Select a level and grid using the controls on the spine of the device.
5. Press and hold the record button until the record light activates, then release.
6. Press and hold the desired message location, begin speaking when you hear a beep. Release when finished.
7. Press and release the record button to exit record mode.
8. Your device is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.


Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

3 Device Overview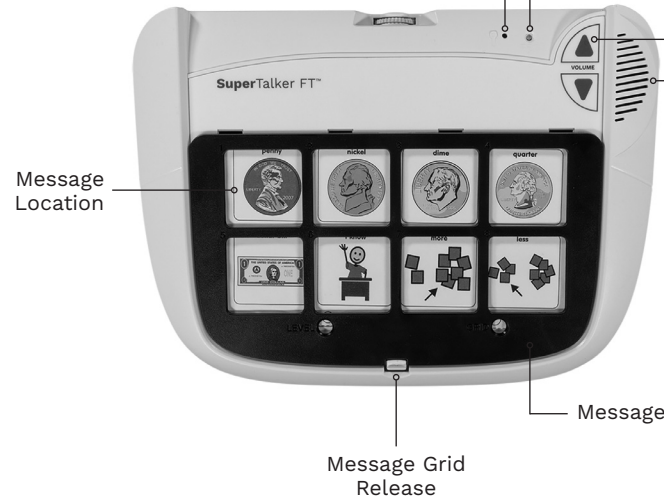

Microphone

Recording Indicator Light

Volume Control Buttons


Sound Output Speaker

Battery Compartment

Message Grid Storage

On/Off Switch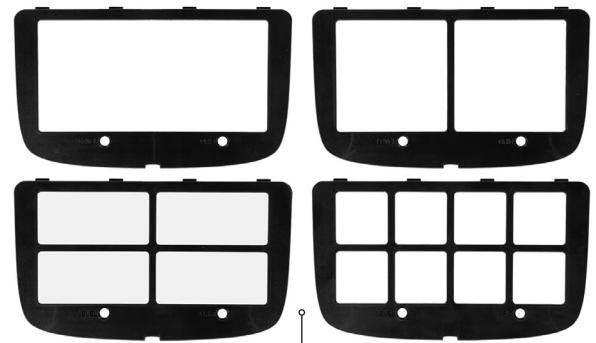

Message Grids (SuperTalker FT includes message grids in 1, 2, 4, and 8-message formats)

Create printable picture symbol overlays for your device with the *FREE* AbleNet Symbol Overlay Maker app. Scan the QR code to download today!

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