QUICK START GUIDE

TrackerPro™ 2

hands-free computer mouse

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 1:59 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Place a TrackerPro 2 reflective dot on forehead of user.
2. Place TrackerPro 2 directly in front of the user.
3. Plug TrackerPro 2 into available USB A port on device. The Power/Status LED on TrackerPro 2 will turn red.
4. Position user 12-in to 36-in (30-cm to 91-cm) away from TrackerPro 2.
5. The Power/Status LED on TrackerPro 2 will turn green once it detects the reflective dot on the user.
6. Position the mouse cursor in the center of the device screen before the user starts.
7. Use small head movements to move the mouse cursor on the screen of the device.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

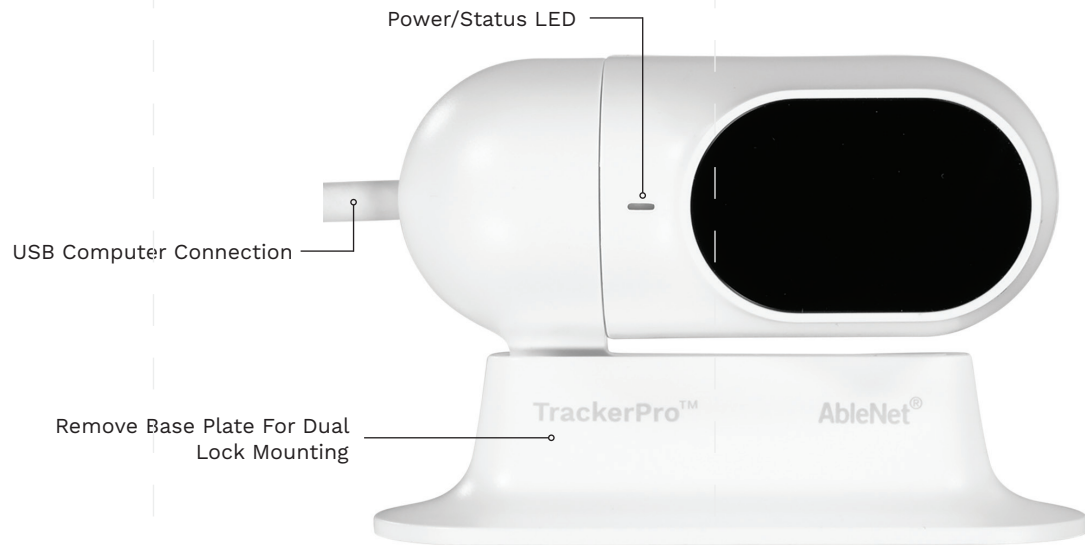
ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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