QUICK START GUIDE

BIGtrack Wireless wireless computer mouse

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 1:55 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started - Bluetooth:

1. Insert two alkaline AA batteries into BIGtrack Wireless.
2. On the front of BIGtrack Wireless, move mode switch to left to select Bluetooth mode.
3. Select and pair the BIGtrack Wireless within the Bluetooth settings of your device. Please consult the Bluetooth pairing process for your device as needed.
4. Your mouse is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

To get started - USB Wireless Receiver:

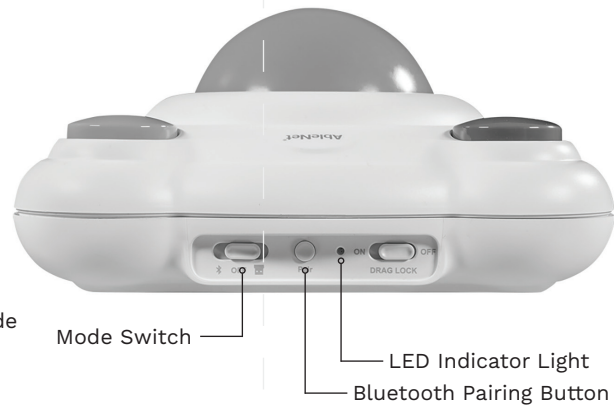
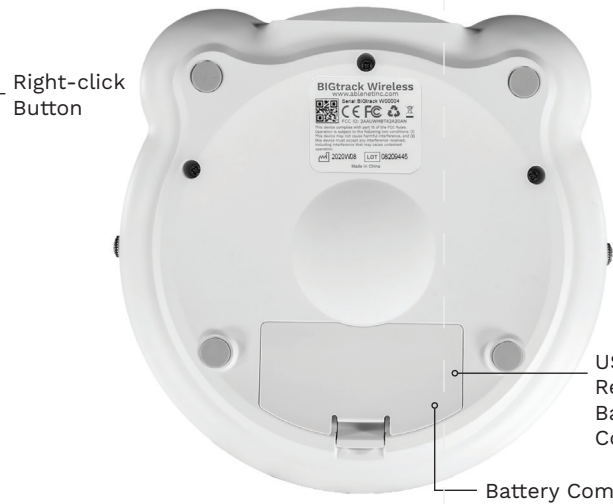
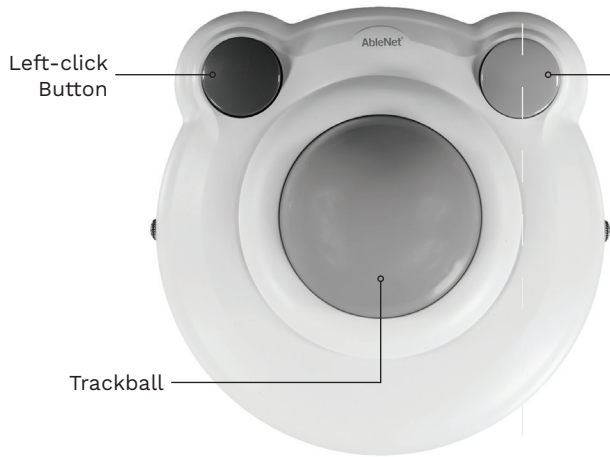
1. Insert two alkaline AA batteries into BIGtrack Wireless.
2. On the front of BIGtrack Wireless, move mode switch to the right to select USB wireless mode.
3. Remove USB wireless receiver from battery compartment and plug it into a USB A port on your device.
4. Your mouse is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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