QUICK START GUIDE

BIG Step-by-Step™

speech device

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 1:54 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Add one alkaline 9-volt battery to device (battery not included).
2. Turn device on.
3. Select desired level.
4. Press and hold record button until recording indicator light turns on.
5. Press and hold colored switch top. Begin speaking into microphone after you hear beep.
6. When done recording message, release colored switch top.
7. Repeat steps 5 and 6 to record additional messages.
8. Press the record button to turn the recording indicator light off.
9. Optional: Select a different recording level to record a second and third set of messages.
10. Adjust volume as needed.
11. Your device is ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.



ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

3 Device Overview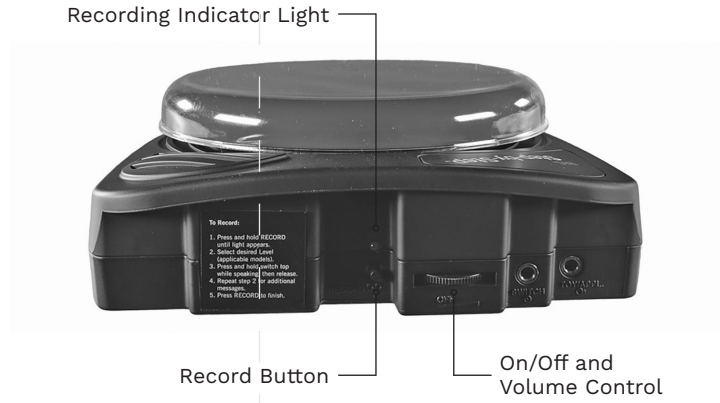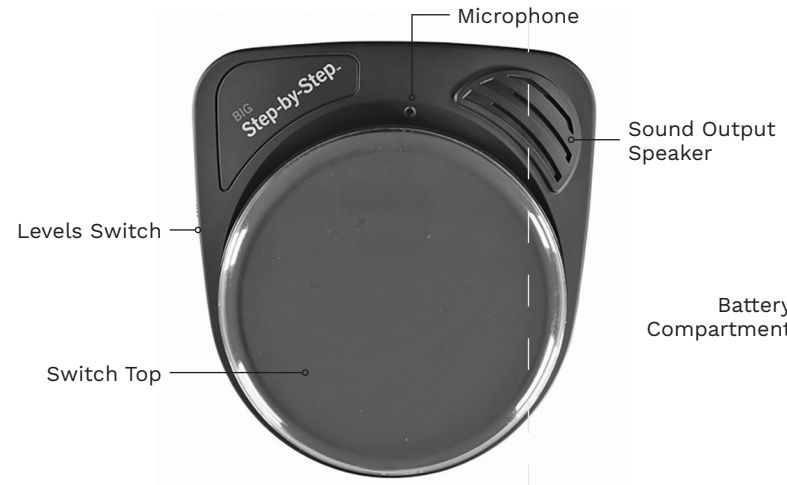

Create printable picture symbol overlays for your device with the *FREE* AbleNet Symbol Overlay Maker app. Scan the QR code to download today!

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