QUICK START GUIDE

Blue2™ FT

wireless accessibility switch

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN February 28, 2023 2:53 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. On an iPad or iPhone go to Settings App > Bluetooth and make sure Bluetooth is turned on.
2. Turn Blue2 FT on.
3. Select BLUE2 FT XXXXX (X's denote Blue2's serial number). If more than one Blue2 FT is available, the serial number is located on the bottom of the Blue2 FT.
4. The first time you use Blue2 FT the blue switch top (S1) is configured to Space and the yellow switch top (S2) is configured to Enter. Keyboard commands for each switch can be changed to Space, Enter, 1, 2, 3, or 4.
5. Configure Switch Control in the Settings App.
6. Your Blue2 FT is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

ableCARE™

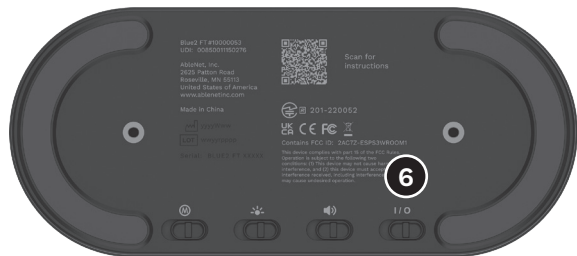
Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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