QUICK START GUIDE

Jelly Beamer™

wireless accessibility switch

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Add two alkaline AA batteries to wireless switch (batteries not included).
2. Turn wireless switch on
3. Turn wireless receiver on and activate pairing mode on the wireless receiver.
4. Activate the switch top on your wireless switch to pair with the wireless receiver
5. Turn pairing mode off on your wireless receiver
6. Your wireless switch is now ready to use with your wireless receiver.

Note: AbleNet wireless switches are only compatible with AbleNet wireless receivers and PowerLink 4.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.



ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

3 Device Overview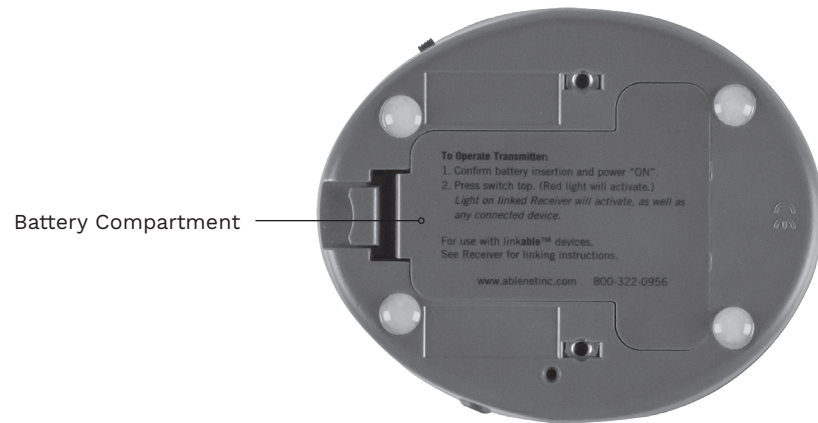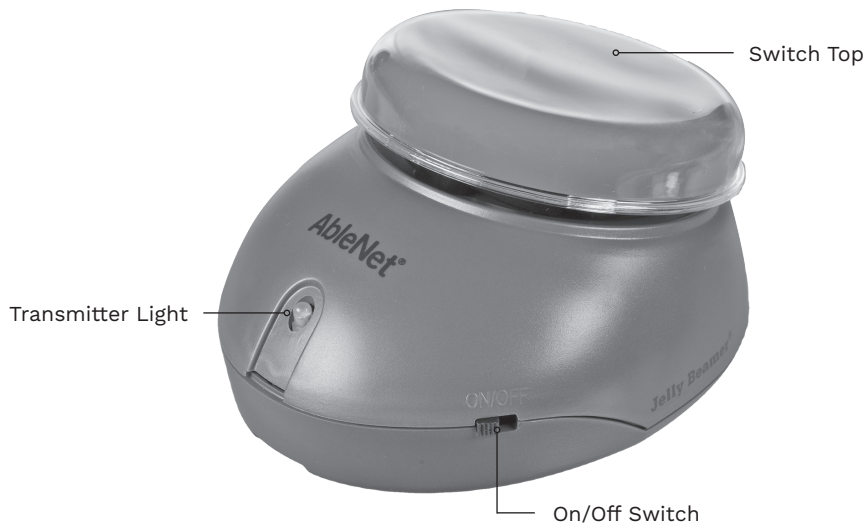

Create printable picture symbol overlays for your device with the *FREE* AbleNet Symbol Overlay Maker app. Scan the QR code to download today!

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