QUICK START GUIDE
BigBlu VisionBoard™
Bluetooth keyboard

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 1:45 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Remove the battery compartment door by removing the screw.
2. Insert two alkaline AAA batteries and replace the battery compartment door and screw.
3. Turn the on/off switch on. The LED indicator light will blink blue once.
4. Press the pairing button to enable the keyboard pairing mode. The LED indicator light will blink blue until the keyboard is paired or until 2-minutes has passed.
5. Select and pair the keyboard within the Bluetooth settings of your device. Please consult the Bluetooth pairing process for your device as needed.
6. The LED indicator light will stop blinking after completing the pairing process.
7. Your keyboard is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

ableCARE™

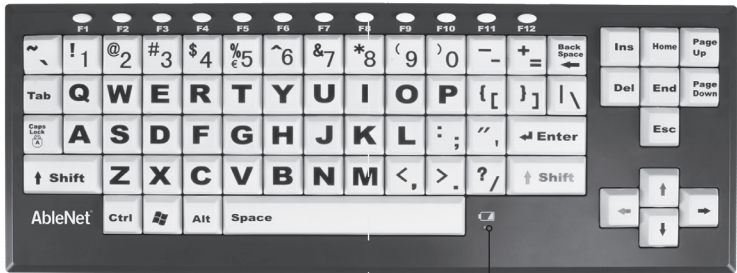
Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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