AbleNet[®]

QUICK START GUIDE

Duo[™]

wireless computer mouse

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content

EN June 18, 2021 1:46 PM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started - Bluetooth:

1. Insert two alkaline AAA batteries into Duo wireless mouse.
2. Remove the USB wireless receiver from the bottom of the Duo wireless mouse. The USB wireless receiver is not used with a Bluetooth connection, but would be used if a device does not have Bluetooth.
3. Open Bluetooth settings on device you are pairing mouse to.
4. Move Duo wireless mouse mode switch towards the Bluetooth icon to turn on Bluetooth mode.
5. Activate Bluetooth pairing button on Duo wireless mouse for 5-seconds and then release.
6. On your device, select ANDUO from the list of Bluetooth devices available.
7. Your mouse is ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

To get started - USB Wireless Receiver:

1. Insert two alkaline AAA batteries into Duo wireless mouse.
2. Remove the USB wireless receiver from the bottom of the Duo wireless mouse and insert it into a USB A port on your device.
3. Move the mode switch towards the USB receiver icon to turn on USB wireless mode.
4. Your mouse is now ready to use.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

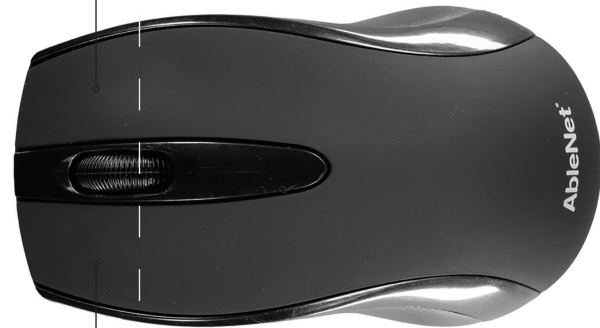
Scan the QR code below to download the free ableCARE app to your phone or tablet.

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