



Instructions For Use

Quick Ready Tablet Holder

Issue Date: February 26, 2024
Revision: 1.0

Model: 80000119
UDID: 00850011150214

 <p>AbleNet, Inc. 2625 Patton Road Roseville, MN 55113 United States of America www.ablenetinc.com</p>	 <p>EUCEREP Roald Dahllaan 33, 5629MC Eindhoven The Netherlands eucerep@eucerep.com</p>	<p>UK Responsible Person</p> <p>EUCEREP LTD 7 Tibbets Close, Meeting Lane Alcester, Warwickshire B49 5QU England, United Kingdom</p>
---	--	--

 <p>MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug Switzerland Telephone: +41 41 562 01 42</p>

INTENDED PURPOSE

The Quick Ready Tablet Holder is for most tablets. It is to help the client maintain their device in an ideal position for greatest ease and accuracy.

CLINICAL BENEFITS OF THE DEVICE

Helps the client maintain their device in an ideal position for greatest ease and accuracy. This includes Assistive Speech Devices.

PERFORMANCE CHARACTERISTICS AND INFORMATION FOR HEALTH CARE PROFESSIONAL

Tablet Dimension Range for Holder:

Height Range: 9" - 11.5" (22.86 cm - 29.2 cm)

Width Range: 6.38" - 8.31" (16.2 cm - 21.1 cm)

Depth Range: 0.13" - 1.12" (0.33 cm - 2.8 cm)

Connection - ¼-20 Male

Overall Dimensions: 9 x 8.875 x 3.5" (22.86 x 22.54 x 8.89 cm)

Weight: 10.6 ounces (300g)

Material: Tablet Holder: Poly Carbonate

Hardware components - Stainless Steel

Suggested Operating Environment:

Ambient Temperature Range: 32°F - 95°F (0°C - 35°C)

Humidity: 10% - 95% (no condensation on device)

Suggested Transportation and Storage:

Ambient Temperature Range: -25°F - 158°F (-31°C - 70°C)

Humidity: 10% - 95% (no condensation on the device)

RISKS, WARNINGS, AND CONTRA-INDICATIONS

Device is not a toy and should not be thrown.

DEVICE OVERVIEW

1	Threaded Bolt
2	Locking Knob



USE INSTRUCTIONS

Compatibility

The Quick Ready Tablet Holder is compatible with all AbleNet mounting arms or any mounting arm that uses a ¼”-20 threaded insert to attach a mounting plate.

First Time Use

1. Attach the threaded bolt on the Quick Ready Tablet Holder to the end of the mounting arm.
2. Turn the Locking Knob on the Quick Ready Tablet Holder clockwise to tighten.
3. Insert tablet into the Quick Ready Tablet Holder.

PREPARATORY TREATMENT

Cleaning and disinfecting your AbleNet assistive technology devices between uses is essential to the health and safety of the user. To help fight against the spread of germs, here are some helpful steps to cleaning and disinfecting your AbleNet device.

Always be sure to wear the appropriate personal protective equipment such as protective gloves and safety glasses.

Cleaning Your Device

Before disinfecting your device, remove any dirt or debris that may be on the device.

1. Turn the device off and remove the batteries.
 1. Remove any large chunks, visible heavy dirt, or debris that is attached to the device.
2. Wipe down the device with cleaning wipes.
3. Use a cotton swab soaked with cleaner to get into any tight areas that were not accessible with the cleaning wipes.
4. Let the device completely dry.

Disinfect Your Device

Now that the device is clean, it is time to disinfect it. Changing protective gloves may be required before disinfecting the device to ensure no dirt or debris is transferred back to the device.

1. Make sure the device is still turned off and that the batteries are removed.
2. Use a clean towel or wipes with Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, or Fungicidal to disinfect the surfaces of the device. It is important to note that these chemicals only disinfect when they are wet and should be left wet on the device for at least 4-minutes.
3. Use a cotton swab soaked with Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, or Fungicidal to disinfect the tight areas that were not accessible with the towel or wipes. Make sure the chemical stays wet on the device for at least 4-minutes.
4. Let the device air-dry entirely before using it.

TROUBLESHOOTING

Weight of tablet opens the jaws of the Tablet Holder

1. Verify that the moveable part of the tablet holder is at the top for 'portrait' holding of the tablet.

PREVENTATIVE AND REGULAR MAINTENANCE

Clean the Quick Ready Tablet Holder if necessary. No scheduled maintenance is required.

REUSABLE

Quick Ready Tablet Holder should be sent back to the manufacturer for cleaning, testing and refurbishment before reuse with another patient.

DEVICES USED TOGETHER WITH

- Mounting system - example AbleNet's Cling!
- Tablet based communication aid - example - QuickTalker Freestyle

PROPER DISPOSAL

Contact your local recycler for instructions on where to recycle the materials in Quick Ready Tablet Mount.

WHEN TO CONTACT A HEALTHCARE PROFESSIONAL

If you have difficulty accessing the switches or need assistance with product placement.

Change in your condition.

NOTICE OF SERIOUS INCIDENT

Notice - If any serious incident has occurred while using this device, seek medical attention and then - please report to AbleNet, Inc and your competent authority of the member state in which you are in.