## QUICK START GUIDE

### Cling mounting arm

#### WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at [www.ablenetinc.com](http://www.ablenetinc.com).

AbleNet, Inc.  
2625 Patton Road  
Roseville, MN 55113  
United States of America  
(651) 294-3101  
[ablecare@ablenetinc.com](mailto:ablecare@ablenetinc.com)  
[www.ablenetinc.com](http://www.ablenetinc.com)

Made with recycled content

EN October 15, 2021 2:24 PM

## 1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

## 2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

### To get started:

1. Place suction cups on a flat, non-porous surface. Move the lever on each suction cup to the lock position to secure it to the flat, non-porous surface.
2. Attach threaded bolt side of mounting plate to end of mounting arm.
3. Use the locking knob to adjust the mounting arm. Turning the locking knob counterclockwise to loosen and then clockwise to tighten after positioning the mounting arm.

*To request a different mounting arm, clamp, or mounting plate, contact ableCARE at [ablecare@ablenetinc.com](mailto:ablecare@ablenetinc.com) or (651) 294-3101.*

*This device can do more! Full Instructions for Use are available at [www.ablenetinc.com](http://www.ablenetinc.com).*

ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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