## QUICK START GUIDE

### PowerLink® 4 power control unit

#### WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at [www.ablenetinc.com](http://www.ablenetinc.com).

AbleNet, Inc.  
2625 Patton Road  
Roseville, MN 55113  
United States of America  
(651) 294-3101  
[ablecare@ablenetinc.com](mailto:ablecare@ablenetinc.com)  
[www.ablenetinc.com](http://www.ablenetinc.com)

Made with recycled content

## 1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

## 2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

### To get started:

1. Plug the PowerLink into any AC wall outlet with the cord provided and turn the device on.
2. Plug appliance(s) to either power cable on the side of the PowerLink and turn the appliance(s) on.
3. Test your AC power connection by pressing the appropriate test button. The attached appliance should activate.
4. Attach any corded, single switch(es) to the corresponding switch jack(s).
5. Press the mode button until the “direct” mode LED lights up.
6. Press and hold switch(es) to activate their corresponding appliances. Release to turn off.

*This device can do more! Full Instructions for Use are available at [www.ablenetinc.com](http://www.ablenetinc.com).*

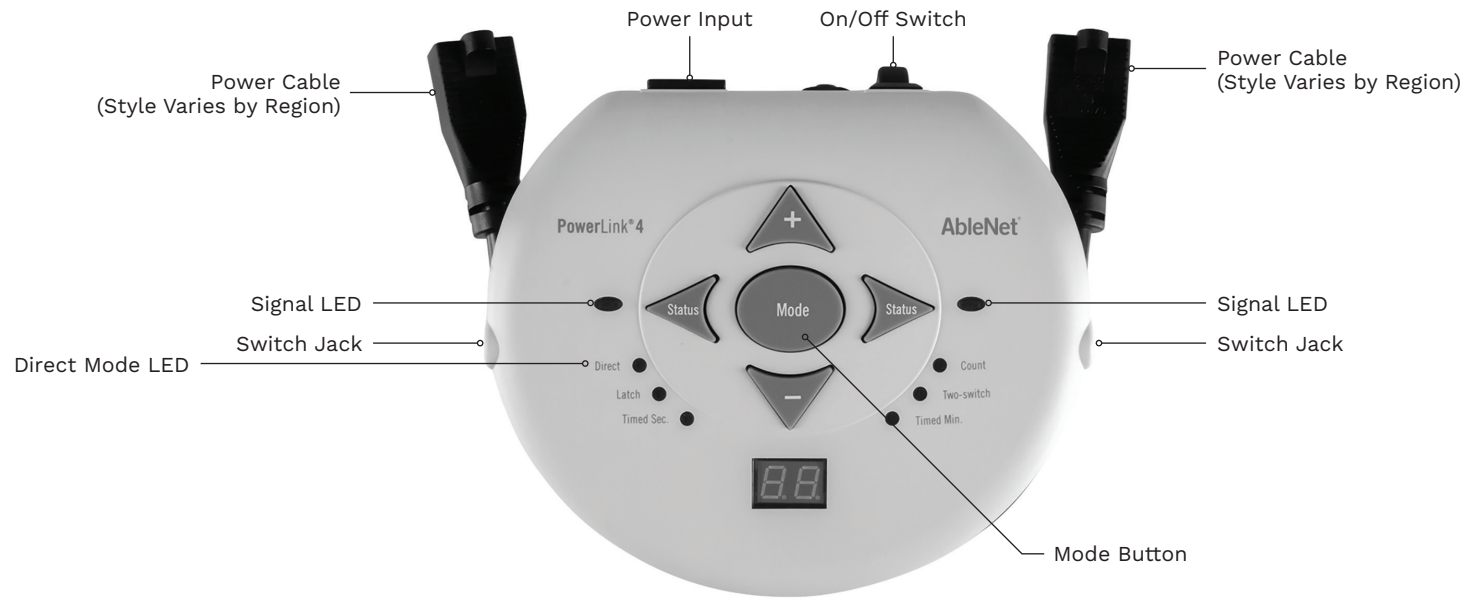
ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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