## QUICK START GUIDE

# QuickTalker™ FT 7/12/23

speech device

### WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at [www.ablenetinc.com](http://www.ablenetinc.com).

AbleNet, Inc.  
2625 Patton Road  
Roseville, MN 55113  
United States of America  
(651) 294-3101  
[ablecare@ablenetinc.com](mailto:ablecare@ablenetinc.com)  
[www.ablenetinc.com](http://www.ablenetinc.com)

Made with recycled content

EN June 18, 2021 2:13 PM

## 1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

## 2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

### To get started:

1. Add four alkaline AA batteries to device (batteries not included).
2. Add a printed picture symbol overlay to device (not included).
3. Turn device on.
4. Select recording level.
5. Press and hold the record button until the recording indicator light turns on, then release the record button.
6. Press and release the desired message location. Begin speaking into microphone to record message. Press and release the desired message location a second time to stop recording.
7. Repeat step 6 for other message locations.
8. Press record button to turn the recording indicator light off and exit record mode.
9. Your device is now ready to use.

*This device can do more! Full Instructions for Use are available at [www.ablenetinc.com](http://www.ablenetinc.com).*


ableCARE™

Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

# 3 Device Overview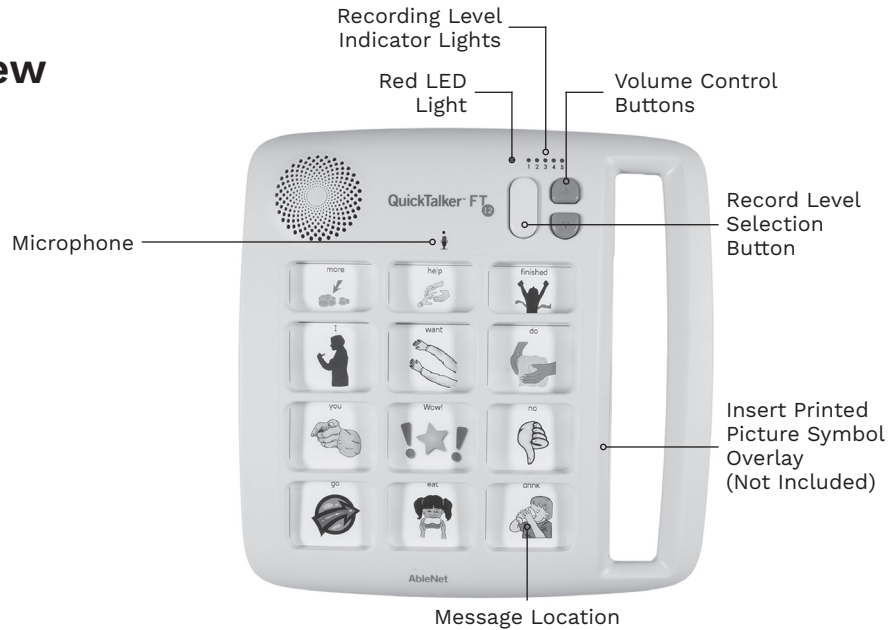

Create printable picture symbol overlays for your device with the *FREE* AbleNet Symbol Overlay Maker app. Scan the QR code to download today!

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